evo

SportNeb™

Nebulizer Compressor

3050

CAUTION: USA Federal law restricts this device to sale by or on the order of a physician.

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P/N 21392 Rev. C

Table of Contents

Specifications	2
Introduction and IEC Glossary	3
Important Parts of the SportNeb	4
Operating Instructions	5
Maintenance	6
Compressor Cleaning	6
Nebulizer Cleaning	7
Filter Change	8
Safety Information	9
Warranty	10

SPECIFICATIONS

SIZE

7.5" L X 4.5" W X 7.5" H
19 cm x 11.5 cm x 19 cm

WEIGHT

3.75 lbs. (1.7 kg) 110V

ELECTRICAL REQUIREMENTS

120V AC 60HZ 1.8A

OPERATING TEMPERATURE

Ambient conditions
(min 50°F to max 104°F)
(min 10°C to max 40°C)

STORAGE CONDITIONS

Room temperature preferred
(min -4°F to max 140°F)
(min -20°C to max 60°C)

EVO Medical Solutions
2636 289th Place
Adel, IA 50003
515.993.5001
515.993.4172 f
800.759.3038 (toll free)

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Your physician has prescribed a liquid medication to treat your respiratory condition. In order to ensure a high quality aerosol nebulization he/she has prescribed the SportNeb from eVO. The SportNeb uses compressed air to turn your liquid medication into an aerosol mist that penetrates your lungs.

To maximize your benefit from the SportNeb system, we recommend you read and understand the information contained within this instruction guide. By following your doctor’s advice and these simple instructions, your SportNeb system will become a valuable part of your daily treatment routine.

IMPORTANT SAFEGUARDS

Follow all basic safety precautions. Please read instructions before using. Precautions are not limited to the safety information provided. Important information is highlighted as indicated:

WARNING

Important safety information for hazards that might cause serious injury.

CAUTION

Important information to prevent damage to the device.

NOTE

Information requiring special attention.

IEC CLASSIFICATION/ GLOSSARY

- Equipment Classification with respect to protection from electric shock: **Class II**
- Degree of protection from electric shock: **Type BF**
- Degree of protection against ingress of liquids: **ordinary**
- Equipment not suitable for use in the presence of flammable anaesthetic mixture with air or with nitrous oxide.
- Mode of operation: **Continuous**

Important Parts to the SportNeb

- 1 External Filter
- 2 Flow Outlet
- 3 Mouthpiece
- 4 Disposable Nebulizer
- 5 Replacement Filters
- 6 Tubing

Note: Actual Disposable Nebulizer or Mouthpiece may differ from photo.

 Type BF equipment
  Class II equipment
  Alternating Current
  Attention: Consult accompanying documents

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- 1 Place the SportNeb on a level, sturdy surface so the controls of the unit can be easily reached when you are seated.
- 2 Pull out the power cord from the recess in the base of the unit.
- 3 Make sure the on/off (1/0) switch is in the off (0) position. Plug cord into the power source.
- 4 Unpack the disposable nebulizer and connect one end of the tubing to the flow outlet.
- 5 Assemble the clean disposable nebulizer by following the instructions packed with the nebulizer.
- 6 Add prescribed liquid medication to the top of the nebulizer using an eye dropper, pre-measured unit dose or ampoule.
- 7 Insert mouthpiece into nebulizer.
- 8 Connect tubing to inlet on the bottom of the nebulizer.
- 9 Press the on/off (1/0) switch to the on (1) position to start the SportNeb compressor.
- 10 Begin treatment by placing nebulizer mouthpiece between the teeth. Close mouth. Inhale deeply and slowly as aerosol begins to flow, then exhale slowly.
- 11 If an aerosol mask is being used, position the mask over the nose and mouth. Inhale deeply and slowly through the mouth as aerosol begins to flow. Exhale slowly.
- 12 Always unplug unit and properly store after each use.

NOTE

Before initial operation, please clean the disposable nebulizer by following the cleaning instructions on page 7. Always wash your hands before you begin the treatment procedure.

NOTE

In high humidity conditions, water build up can occur in the SportNeb tubing. If this occurs, simply turn on the compressor and run it for 2-3 minutes before connecting tubing to the flow outlet.

NOTE

Cold temperatures may result in poor compressor performance. Bring machine up to room temperature for best results.

WARNING

To avoid the risk of possible electrical shock and damage to compressor, SportNeb must be operated on the specified power source.

Maintenance

NOTE

Maintenance manual and technical information with parts list is available through EVO upon request.

WARNING

Do not remove the compressor case/cabinet. Failure to comply could result in electrical shock. All disassembly must be performed by an authorized EVO service center.

WARNING

If unit does not operate normally, discontinue use and contact your representative.

All maintenance on your SportNeb system must be performed by an authorized EVO service center.

Compressor Cleaning

CAUTION

Never submerge the SportNeb compressor in water, hold under running water or spray with water or other liquids. This will result in compressor damage and possibility of electrical shock/electrocution.

- 1 Make sure the on/off (1/0) switch is in the off (0) position.
- 2 Unplug the compressor from the wall outlet.
- 3 Wipe the compressor with a clean, damp cloth every 2-3 days. This will keep the unit dirt and dust free.
- 4 If liquid enters the case, discontinue use and contact your representative.

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- 1 Please refer to the instruction sheet supplied with your disposable nebulizer.
- 2 Make sure the on/off (1/0) switch of the compressor is in the off (0) position.
- 3 Unplug the cord from electrical source.
- 4 Disconnect tubing from the flow outlet on the compressor. Disconnect the tubing from the nebulizer. Set the tubing aside.
- 5 Remove the nebulizer cap by turning counterclockwise.
- 6 Remove nebulizer baffle by grasping and pulling from the chamber.
- 7 After every use rinse all nebulizer parts with hot tap water.
- 8 Soak nebulizer daily in 3 parts hot water to 1 part white vinegar for 30 minutes.
- 9 Rinse under hot tap water and air dry.

NOTE

The disposable nebulizer is designed for single patient use.

WARNING

To prevent possible risk of infection due to contamination, clean the nebulizer after each aerosol treatment. Always use a fresh cleaning solution and do not towel dry the nebulizer components after cleaning.

Your SportNeb compressor comes with a packet of extra inlet filters. Please contact your local dealer to obtain additional filters.

- 1 Check your external filter (refer to page 4) at least once per month. The filter should be changed when it begins to turn brown or gray in color.
- 2 Remove the filter cap by grasping cap and pulling straight out.
- 3 Remove dirty filter and discard.
- 4 Replace with a new filter.
- 5 Replace cap and push into position.

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This EVO product is an electrical device. Always follow basic safety precautions when using this device, including but not limited to the following:

To reduce the risk of burns, fire, electrocution, or other bodily harm or injury to persons:

- 1 Always unplug this device after each use.
- 2 Never submerge this product in water.
- 3 Never use this product while bathing, showering, washing dishes, or around a water source of any kind.
- 4 Never leave this product unattended.
- 5 Use this product only for its intended use as described in the operator manual.
- 6 Closely monitor and supervise use by invalids and children.
- 7 Never block the air vent openings or allow objects to fall or to be inserted into the air vent openings.
- 8 Observe caution when using this product where oxygen is being administered.
- 9 **If this product is being used in a life threatening situation, a backup unit is necessary.**
- 10 Keep the compressor case dry.
- 11 Do not allow liquids to accumulate on case or enter case.
- 12 If liquid enters the case, discontinue use and contact your representative.
- 13 If unit does not operate normally, discontinue use and contact your representative.
- 14 Always unplug unit and properly store after each use.

WARNING
 This equipment is not suitable for use in the presence of a flammable anaesthetic mixture with air or with oxygen or nitrous oxide.

EVO warrants the SportNeb nebulizer compressor to be free from defective workmanship and materials for a period of 5 years from the date of purchase. This warranty is limited to the dealer. Any defective part or assembly will be repaired or replaced, at the sole discretion and determination of EVO if the unit has not been misused or tampered with during the warranty period. Normal maintenance items, as outlined in this manual and disposable components are not covered by this warranty. Shipping charges, if any, shall be paid by the purchaser.

NOTE

There is no other express warranty. Implied warranties, including those of merchantability and fitness for a particular purpose, are limited to the duration of the express limited warranty and to the extent permitted by law and all implied warranties are excluded. This warranty does not cover providing a loaner compressor, compensating for costs incurred for compressor rental, or labor costs incurred in repairing or replacing defective part(s).

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